

ADVANCING INTRAOPERATIVE GONIOSCOPY in a convenient, single-use format

The newly designed **iprism[®]S** provides exceptional clarity with expanded views of all angle structures.

iprismS offers new design features for precise lens handling during intraoperative gonioscopy procedures.




Expanded incision access eliminates device interference at the incision during gonioscopy

Wide angle lens dramatically expands field of view for a broader perspective of all angle structures



Unique concave lens geometry delivers precise coupling with the corneal surface for a quick, clear, and "self-centering" view of the trabecular meshwork

Anti-reflective technology delivers crystal clear views of angle structures without light distortion or interference

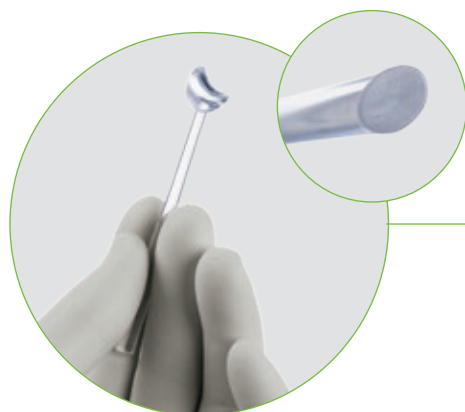

Lightweight, low-profile design fits elegantly in the fingers for unparalleled control and enables lighter touch for reduced corneal striae

iprism[®]*S* is the recommended gonioscopic lens for iStent[®] Trabecular Micro-Bypass procedures



The iprism S is a single-use, sterile device that is conveniently packaged and ready for use. There is no need for cleaning, sterilization, or disinfection. Just dispose of the iprism S after the procedure.

iprism S Specifications

Lens Material: **PMMA**

Image Mag: **1.1X**

Static Field of View: **90°**

Contact Surface Diameter: **8.7mm**

Handle Length: **75mm**

Order Numbers

Right-hand *iprism S*: **SGR5**

Left-hand *iprism S*: **SGL5**

CONTACT YOUR GLAUKOS SALES REPRESENTATIVE TO ORDER.

iprism S DIRECTIONS FOR USE:

See *iprism S* package insert for detailed instructions.

iStent DIRECTIONS FOR USE:

See iStent package insert for detailed instructions.

FOR GONIOSCOPY TRAINING, VISIT GONIOSCOPY.ORG

Courtesy of Wallace L.M. Alward, M.D., Director of Glaucoma Services, University of Iowa

Glaukos Corporation

229 Avenida Fabricante • San Clemente, CA • 92672 • USA
tel 800-GLAUKOS (452-8567) • fax 949-367-9838 • Glaukos.com

Manufactured for Glaukos by

Katena Products Inc.

23175 224th Pl SE Suite C
Maple Valley, WA, 98038, USA
Ph: 1-423-358-7381

